



## Quick Start Guide

- ◇ Make sure the Proxi-Mate unit is plugged in, and / or charged up
- ◇ Make sure the Sensor is properly connected—under bedclothes, on top of mattress
- ◇ Make sure the cables are out of the way and not a trip hazard
- ◇ Make sure you have the correct pager(s) & cap codes are programmed in
- ◇ Settle patient / resident into bed / on chair
- ◇ Press the On / Off button—Proxi-Mate Pro shows 'On' sends update to pager (if set to do so). Standard unit light 'On' LED
- ◇ When patient / resident rises alarm is triggered. Attend to person then reset Proxi-Mate (if still needed)
- ◇ If message on pager 'Status Change—Low Bat' then recharge Proxi-Mate unit

[www.proximate.com.au](http://www.proximate.com.au)